



# Tempro

BPH Thermal Treatment



Innovations for life


**DiREX**  
An **initia** Distribution Partner

# Tempro

## BIPOLAR RF TECHNOLOGY FOR BPH TREATMENT

- Tempro presents an innovative approach to thermal RF treatment. Tempro offers a unique bipolar RF technology which leads to length-dependent treatment, allowing the urologist to treat both small and large prostates with the same applicator. Tempro treatment is safe and effective and does not require cooling systems nor rectal temperature probes.

Tempro offers treatment with pain-relief profiles for the individual patient. The treatment may be performed with high, medium or low heating profiles to maximize patient comfort during treatment. The treatment area is chosen according to the Prostate Urethra Length (PUL), for localized treatment, by applying energy to different rings. Tempro incorporates a unique RF generator which enables bipolar treatment. The device also keeps a log of all important information (treatment time-temperature graphs, patient's details). This information is saved in the device and may be printed at the end of treatment. Tempro treatment procedure is short and simple. The software treatment application is designed with a very user-friendly interface.

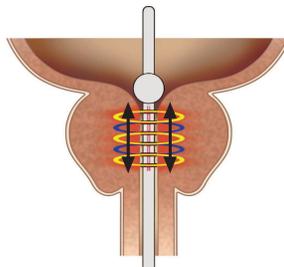

### Unique Features:

The unique features of Tempro include the 6-RING electrode, mounted on a silicone coated 16FR latex foley catheter. Three simultaneous temperature controls measure the highest prostate treatment temperature during treatment. The temperature-time graphs are constantly displayed on the screen, ensuring control over the treatment. Bipolar technology enables concentrated RF transmission to the prostate with minimal energy transmission to surrounding tissue. RF energy used in Tempro allows predictable heat patterns without hot spots. Due to the small diameter of the treatment applicator, its insertion is smooth and painless.

In clinical studies performed with Tempro, 90% of patients were catheter-free post-treatment while the remaining 10 % of patients were catheterized for only 3 days.



*User friendly treatment software with continuous temperature monitoring*

### □ Distinctive Advantages:

- Adjustable treatment length to different prostate sizes
- Anesthesia/sedation free treatment
- In-office procedure, single session
- No need for cooling or rectal monitoring
- Incorporation of a unique bipolar RF energy source
- Low rate of post treatment catheterization
- Silicone coated 16FR latex foley catheter
- Safe and effective procedure

Tempro possesses a unique bipolar radio-frequency energy source. Tempro applicator can be adjusted to different prostate lengths by activating various number of rings according to prostate urethra length. Treatment by Tempro does not need any anesthesia/sedation and is performed in a single session in a doctor's office with analgesia. Tempro device is mobile, easy to operate and is fully equipped with a computerized control and printer for printing of patients' summary forms and files; including post-treatment and statistical analysis of results.



Clinical studies with Tempro shows improvement of irritative and obstructive symptoms.

## Distributors

Initia distributes its products in all markets worldwide. For information about your local distributor, please contact: [info@initiaimed.com](mailto:info@initiaimed.com)

In addition, Initia has a strategic alliance with the DirexGroup distribution network in the following countries:

### North and South America

#### U.S.A. - Direx Systems Corporation

437 Turnpike Street  
Canton, MA 02021  
Tel: +1 339 502 6013  
Toll Free: 888 874 7837  
Fax: +1-339-502-6018  
[marketing@direxusa.com](mailto:marketing@direxusa.com)

#### Mexico - Direx De Mexico, S.A. De C.V

Dr. Jose Ma. Barragan 482  
Col. Atenor Salas  
C.P. 03010, Mexico, D.F  
Tel: + 52-55-5519-7467  
Fax: + 52-55-5519-7102  
[ventas@direxdemexico.com.mx](mailto:ventas@direxdemexico.com.mx)

#### Brasil - Direx Do Brasil Ltd.

Rua Tutoia, 52, Paraiso  
Sao Paulo / SP  
Cep 04007-000 Brazil  
Tel: +55 11 3889 0026  
Fax: +55 11 3884 94370  
[direx@direxdobrasil.com.br](mailto:direx@direxdobrasil.com.br)

#### Argentina - Direx Argentina

San Martin 323 Piso 17  
1004 Buenos Aires, Argentina  
Tel: +54 11 4114 5527  
[info@direx-argentina.com.ar](mailto:info@direx-argentina.com.ar)

### Asia

#### India - Direx Systems Ltd. (U.K.)

Indian Branch  
1 Sivaganga Road  
Chennai 600 034  
Tel: + 91-44-825-7356/7/8  
Fax: +91-44-825-7355  
[direx@vsnl.com](mailto:direx@vsnl.com)

#### Korea - Direx Korea Co. Ltd.

5F Leader's Tower Bld.  
836-70 Yeoksam dong  
Kangnam-gu  
Seoul 135-080  
Tel: +82 2 565 4793 (4794, 4795)  
Fax: +82-2-565-4743  
[bk1231@nate.com](mailto:bk1231@nate.com)

#### Japan - Direx Japan K.K.

Nakagawa Bldg.,4F  
3-31-1 Yushima Bunkyo-Ku  
Tokyo 113-0034, Japan  
Tel: +81 3 5807 3111  
Fax: +81 3 5807 3112  
[contact@direxjp.co.jp](mailto:contact@direxjp.co.jp)

### Europe

#### Germany - Direx Systems GMBH

Blumenstrasse 10  
D - 65189 Wiesbaden  
D - 64349 Reinheim, Germany  
Tel: +800 223 223 22  
Tel: +49 180 215 2152  
Fax: +800 223 223 23  
Fax: +49 180 215 1151  
[europe@Direxgroup.com](mailto:europe@Direxgroup.com)

#### Italy - Direx Systems S.R.L

Via Bernardino Ramazzini 52  
50135 Firenze, Italy  
Tel: + 39-055-667-190  
Fax: + 39-055-667-531  
[direx@fol.it](mailto:direx@fol.it)

DirexGroup is an international network of companies specializing in the distribution of high technology medical equipment for hospitals and private clinics. For the past 20 years, DirexGroup companies have installed thousands of systems in over 70 countries worldwide.

DirexGroup's professional staffs are trained in all companies around the globe to ensure immediate customer support at the highest possible level, both technical and commercial.

DirexGroup companies have entered exclusive agreements to distribute Initia Medical Technologies' devices in the fields of lithotripsy, thermal therapy and radiotherapy. This joint effort between Initia's development and manufacturing capabilities and DirexGroup's well spread market presence enables quick and efficient solutions to our customers.

Please contact your local DirexGroup distributor for further information regarding the device, including local modifications and limitations.

**Regulatory Approval:**  
CE - Europe